EpiMax

94B1/...

Ellenbogenbandage/Elbow support

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Instructions for use

Mode d'emploi

Istruzioni d'uso

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Brugsanvisning

Bruksanvisning

Käyttöohje

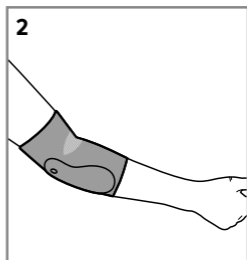
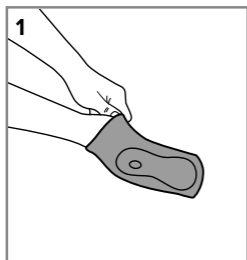
Bruksanvisning

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Upute za uporabu

Návod na používanie

Navodila za uporabo

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EN

Purpose

The EpiMax guides the affected elbow in an anatomically correct way, relieving strain during movement. Medical compressive active supports have a proprioceptive effect. They improve self-perception and can compensate for muscle imbalances. They reduce pain, support the removal of fluids from the tissue, thanks to their rheological effect, and provide relief during movement.

Here is how to put on your EpiMax correctly:

1. Grab the elbow support at its upper, shorter section and pull the support over the lower arm towards the elbow until the oval, patterned area is centred in the crook of the arm.
2. The epicondyles (prominences on the distal part of the elbow) must be located right in the cut-out of the pressure pads. Please adjust the elbow support as required.

The correct fit of your support is guaranteed when it firmly surrounds the elbow without applying too much (potentially painful) pressure on the epicondyles. Fine adjustment of the elbow support may be made at any time by repeating steps 1 and 2 above. Caution: rough, unfilled finger nails or sharp-edged rings etc. may damage the knitted fabric.

Please note:

Only wear your support when prescribed by your physician. Medical aids should only be dispensed by

suitably trained staff of a medical products supplier. The effectiveness of the support can only be realized if it is applied correctly. The support is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it be examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the support yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the support is not suitable for treating more than one patient. The sewn-in textile label is important for the identification and traceability of the product. We therefore ask you not to remove the label under any circumstances.

Washing and care instructions

Please note the care instructions on the textile label sewn into your stockings. Wash the supports separately the first time (colour may run). Your support should be washed every day on a gentle or delicate wash cycle (30° C). Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener! When hand washing, rinse well and do not wring. We recommend the gentle Juzo special detergent. You can reduce drying time by placing the support on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang the support up loosely. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Juzo supports are not suitable for dry cleaning.

Material used

Details are provided on the label sewn into your wrist support.

Storage information and shelf life

Store in a dry place and protect from sunlight. Supports have a maximum shelf life of 42 months. The usage period of the medical product is printed on the box with an hourglass symbol. The medical retailer or doctor prescribing the orthosis will advise on exactly how long it can be used for. Body measurements may change based on the patient's condition and individual circumstances. We therefore recommend that medical specialists check body measurements regularly.

Indications

Lateral and medial epicondylitis (tennis/golfer's elbow), joint effusions and swelling due to osteoarthritis and arthritis, dislocated elbow, chronic, post-traumatic or postoperative irritation of the soft tissue in the area of the elbow joint, post-traumatic pain, sprains (twists/strains).

Contraindications

If you have any of the following conditions, you should only wear the support after consulting with your doctor: Lymph drainage disorders and non-specific

soft tissue swelling outside the area of application, sensory disturbances and impaired circulation in parts of the body to be treated, skin conditions, skin conditions, skin irritation and injuries in the area to be treated; open wounds in the treatment area must be covered with sterile material. Liability for failure to comply with the contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your symptoms worsen during the time you are wearing the product, please take off the support and consult a medical specialist immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the event of complaints related to this product, such as damage to the material or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your support can be disposed of through normal waste channels. No special waste disposal requirements apply.

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**Hersteller/Manufacturer:** Julius Zorn GmbH
CH REP Berro AG · Austrasse 7 · 9055 Bühler · Suisse
UK responsible person: Juzo UK Ltd
**Vertrieb/Distributor:** Streifeneder · ortho.production GmbH